

INSTRENEW® SHARPENING ASSISTANT


InstRenew is the only instrument sharpening device developed by instrument makers to make it EASY FOR ANYONE to sharpen ANY BRAND and ANY PATTERN of Scalers and Curettes with professional results every time. The patented design makes it possible to achieve perfect factory blade angles without the guesswork and frustration that are common with other sharpening methods. If you have been looking for a cost effective, fast and easy way to improve your in-house sharpening program that will get everyone in your practice sharpening with the exact same angle every time...then InstRenew is the answer! See a complete video demonstration at www.nordent.com.

InstRenew Sharpening Assistant comes complete with everything you'll need to begin sharpening with professional results today. Includes:

- Diamond Coated Sharpening Cone (installed)
- Wrench with Diamond Smoother
- Plastic Test Stick
- 110v US - Prong Power Cord
- InstRenew Training DVD



InstRenew Sharpening Stone

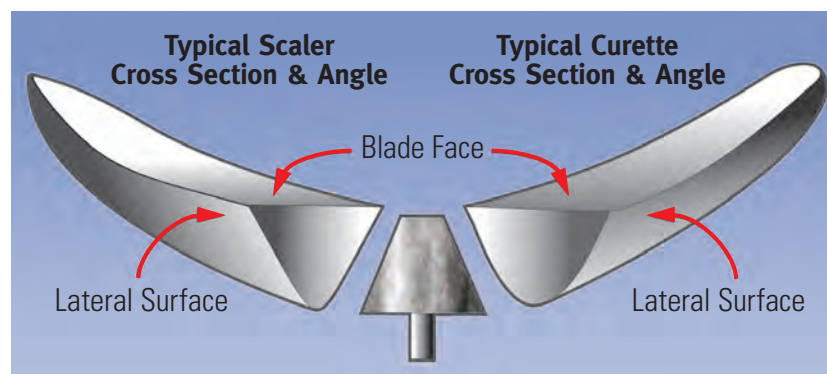

With InstRenew you focus on sharpening the blades and forget about positioning the shanks. Every Brand and Every Pattern of Scalers and Curettes have one thing in common. The angle between the blade face and the lateral surface, that creates the sharp edge, is always the same.

3-Step Method



Position the Blade

The Blade Positioner automatically locks EVERY instrument into the correct position for sharpening by holding the blade face horizontal.



Grasp the Tip

The Tip Clamp allows you to easily grasp the instrument on any straight portion of the shank and maintain the correct position while you sharpen.



Sharpen the Blade

The Sharpening Cone is cut at the correct angle for EVERY brand and EVERY pattern of scalers and curettes. Simply trace each cutting edge over the sharpening cone.